



# AIR-FLOW MASTER Piezon®


# AIR-FLOW MASTER Piezon®

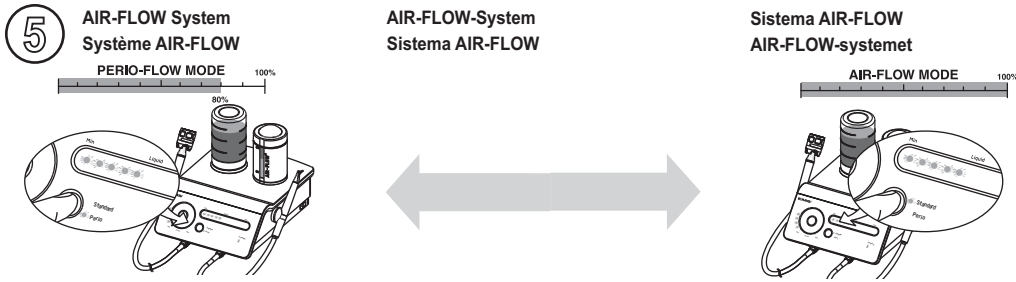



**5** AIR-FLOW System / Système AIR-FLOW / Sistema AIR-FLOW  
**5a** Multifunction foot pedal / Pédale multifonctions / Multifunktionsfußschalter / Multifunction pedal

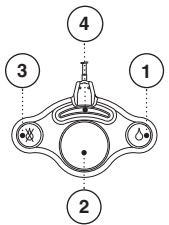

Standard mode	EN	FR	DE	ES	IT	SV
1	Air/Water spray	Jet d'Air/Eau	Luft-/Wasser-Strahl	Pulverización de aire/agua	Spruzzo aria/acqua	Luft-/vattensprej
2	Air/Water/Powder spray	Jet d'Air/Eau/Poudre	Luft-/Wasser-/Pulver-Strahl	Pulverización de aire/agua/polvo	Spruzzo aria/acqua/polvere	Luft-/vatten-/pulversprej
3	Air only	Air seul	Nur Luft	Sólo aire	Solo aria	Endast luft
4 (+2)	Boost function	Fonction "Boost"	Verstärkungsfunktion	Función incremento	Funzione Boost	Boost-funktion
Perio mode						
1 2 3	Same as Standard mode	Identique au mode Standard	Identisch mit Standard-Modus	Igual que el modo Standard	Come in modalità Standard	Samma som Standard-läge
4	Beep On/Off	Beep On/Off	Piepton ein/aus	Pitido activado/desactivado	Segnale acustico on/off	Ljudsignal på/av

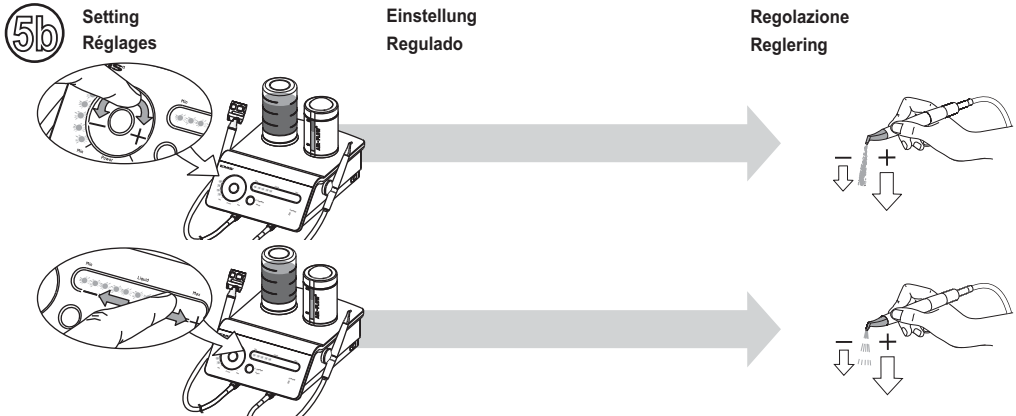

- Correct settings are indicated in the AIR-FLOW or PERIO-FLOW systems operation instructions.
- Les réglages corrects sont indiqués dans les modes d'emplois des systèmes AIR-FLOW ou PERIO-FLOW.
- Die korrekten Einstellungen sind in der Bedienungsanleitung des AIR-FLOW oder des PERIO-FLOW Systemen angegeben.
- Los valores de ajuste adecuados se indican en las instrucciones de empleo de los sistemas AIR-FLOW o PERIO-FLOW.
- Le impostazioni corrette sono indicate nelle istruzioni per l'uso dei sistemi AIR-FLOW o PERIO-FLOW.
- Rätt inställningar respektive anges är bruksanvisningen AIR-FLOW eller PERIO-FLOW system.

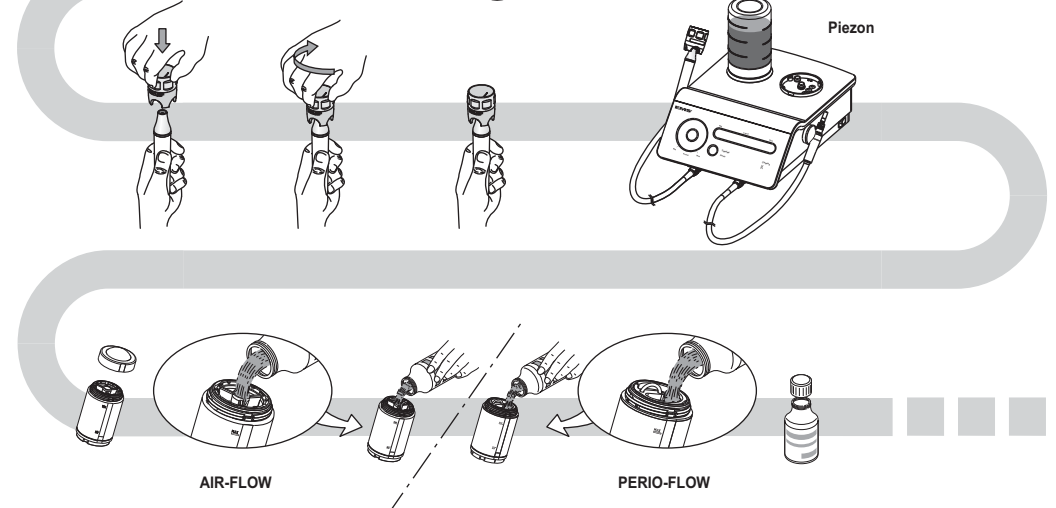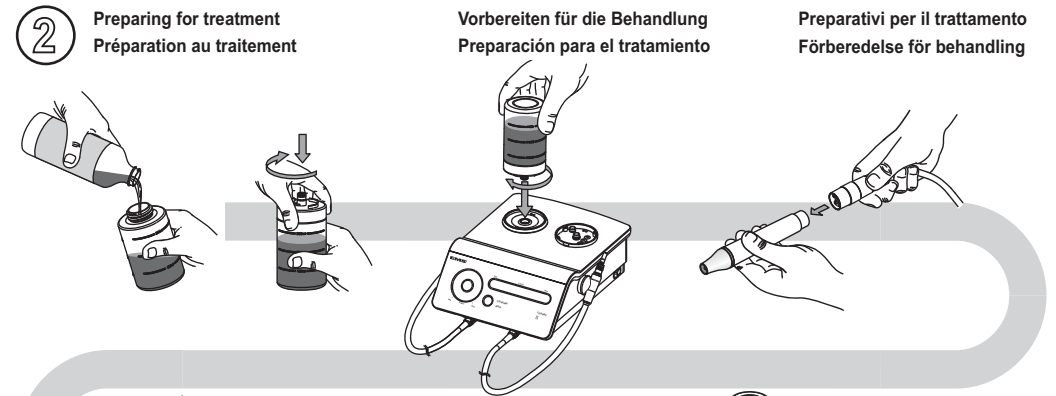


© Copyright EMS SA FB-452/3 ed. 2009/07

Read the operating instructions  
 Consulter le mode d'emploi  
 Bitte Bedienungsanleitung lesen



Ver instrucciones de empleo  
 Vedi istruzioni per l'uso  
 Läs bruksanvisningen

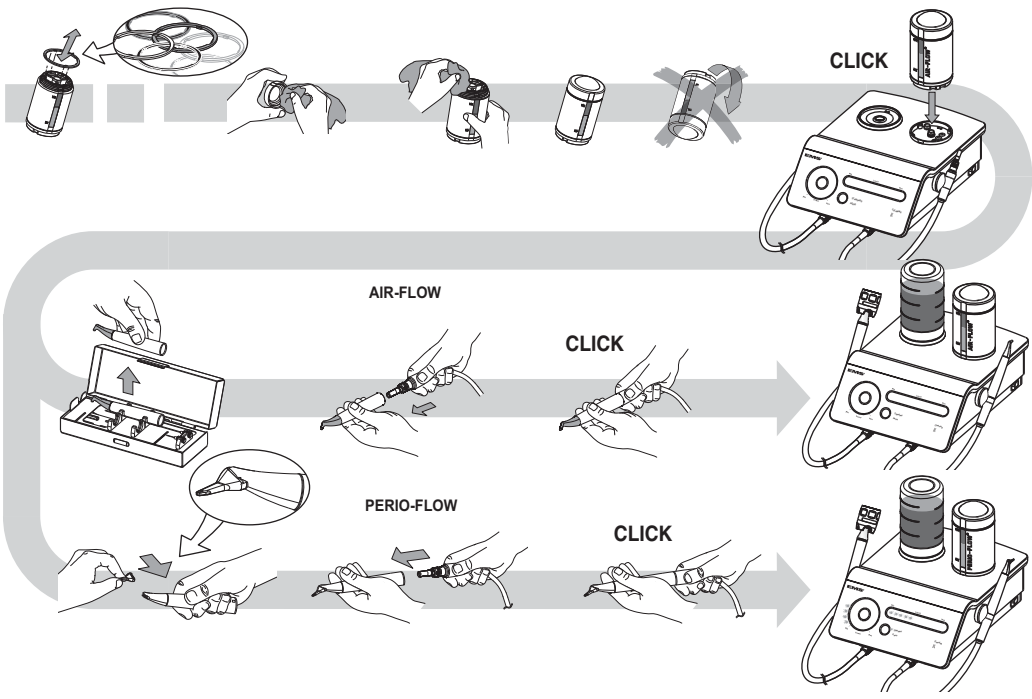


3a


Multifunction foot pedal  
Pédale multifonctions

Multifunktionsfußschalter  
Multifunction pedal

Multifunction pedale  
Multifunction pedal

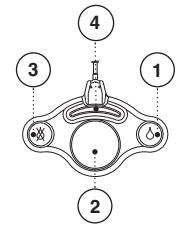


Standard mode	EN	FR	DE	ES	IT	SV
1	Irrigation + light	Irrigation + lumière	Flüssigkeit + Licht	Irrigación + luz	Irrigazione + illuminazione	Spolning + belysning
2	Ultrasounds + irrigation + light	Ultrasons + irrigation + lumière	Ultraschall + Flüssigkeit + Licht	Ultrasonidos + irrigación + luz	Ultrasuoni + irrigazione + illuminazione	Ultraljud + spolning + belysning
3	Ultrasounds + light (DRY WORK mode)	Ultrasons + lumière (mode DRY WORK)	Ultraschall + Licht (Modus "DRY WORK")	Ultrasonidos + luz (modo "DRY WORK")	Ultrasuoni + illuminazione (Modalità "DRY WORK")	Ultraljud + belysning ("DRY WORK"-läge)
4 (+2)	Boost function + light	Fonction "Boost" + lumière	"Boost" Funktion + Licht	Función "Boost" + luz	Funzione boost + illuminazione	Effektökning + belysning
Perio mode						
1 2 3	Same as Standard mode	Identique au mode Standard	Identisch mit Standard-Modus	Igual que el modo Standard	Come in modalità Standard	Samma som Standardläge
4	Active	Active	Aktiv	Activo	Attivo	Aktivt

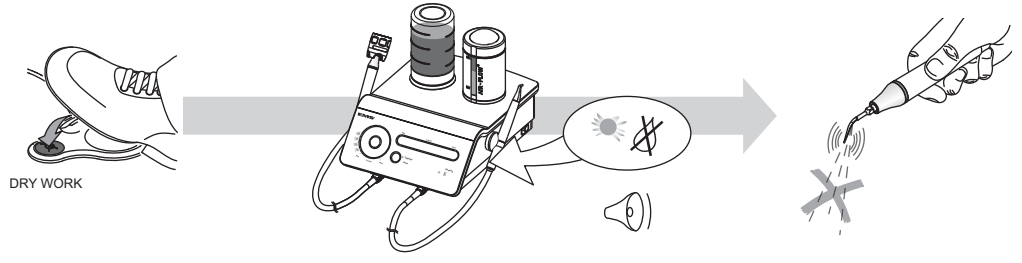

3 Piezon system  
Système Piezon

Piezon-System  
Sistema Piezon

Sistema Piezon  
Piezon-system

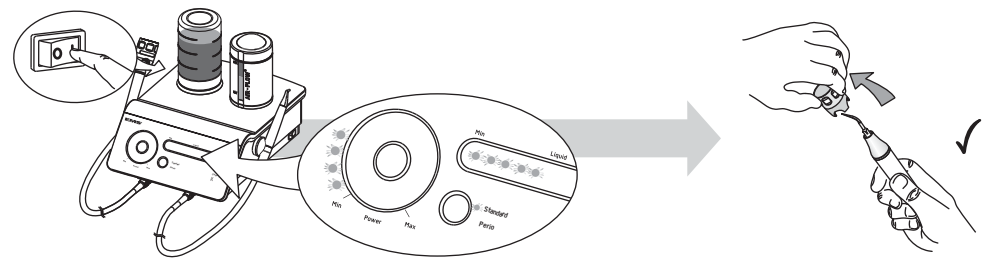


Working mode Mode de travail Arbeitsmodus Modo de trabajo Modalità operativa Driftsläge	Handpiece used Pièce à main utilisée Verwendetes Handstück Mango utilizado Manipolo utilizzato Handstycke som används	Power settings according to the instruments Réglages de puissance selon les instruments Leistungseinstellung je nach Instrument Ajustes de potencia según los instrumentos Regolazione della potenza in base agli strumenti Effektinställning enligt instrumenten	Available power Puissance disponible Verfügbare Leistung Potencia disponible Potenza disponibile Tillgänglig effekt
Standard	Universal handpiece		
Standard	Power handpiece (optional)		
Perio	Universal handpiece Power handpiece (optional)		

3b

Power and flow rate  
Puissance et débit

Leistung und Durchflussrate  
Potencia y caudal

Potenza e velocità del flusso  
Effekt och flöde

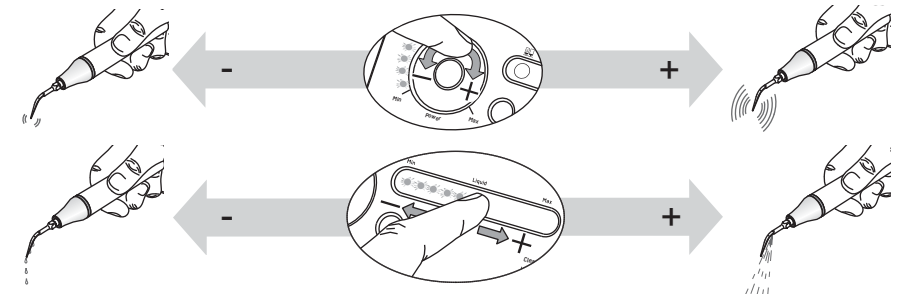

3c

Light  
Lumière

Licht  
Luz

Illuminazione  
Belysning

