OPERATION INSTRUCTIONS

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
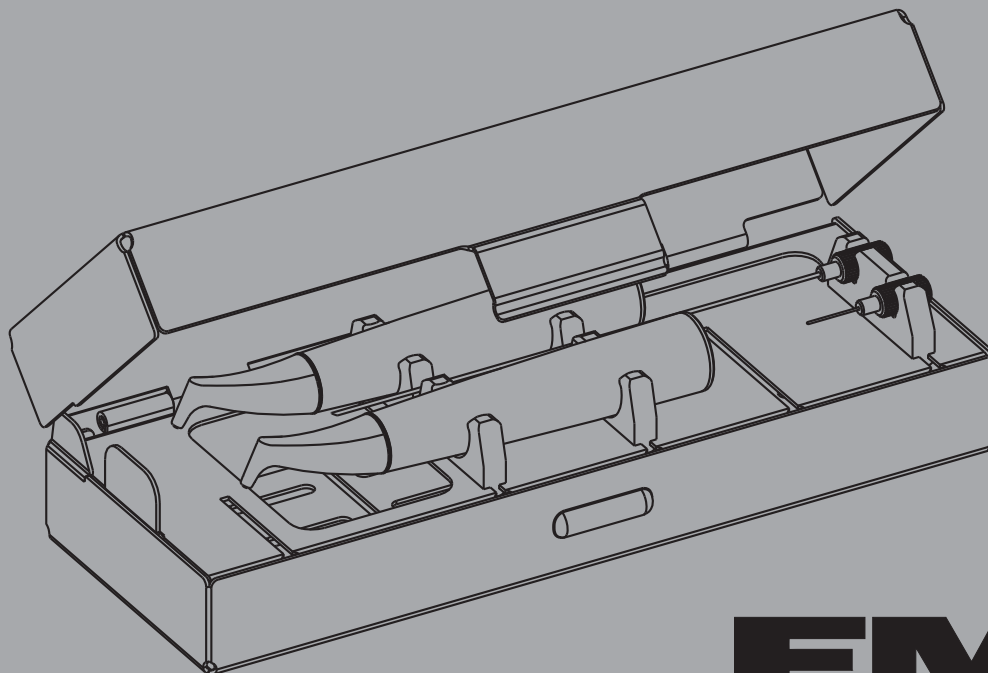
KULLANMA KILAVUZU

AIR-FLOW[®] Prophylaxis Systems

ADDITIONAL SAFETY ADVICE FOR USA:
Caution! Federal USA law restricts this product to sale by or on the order of a dentist

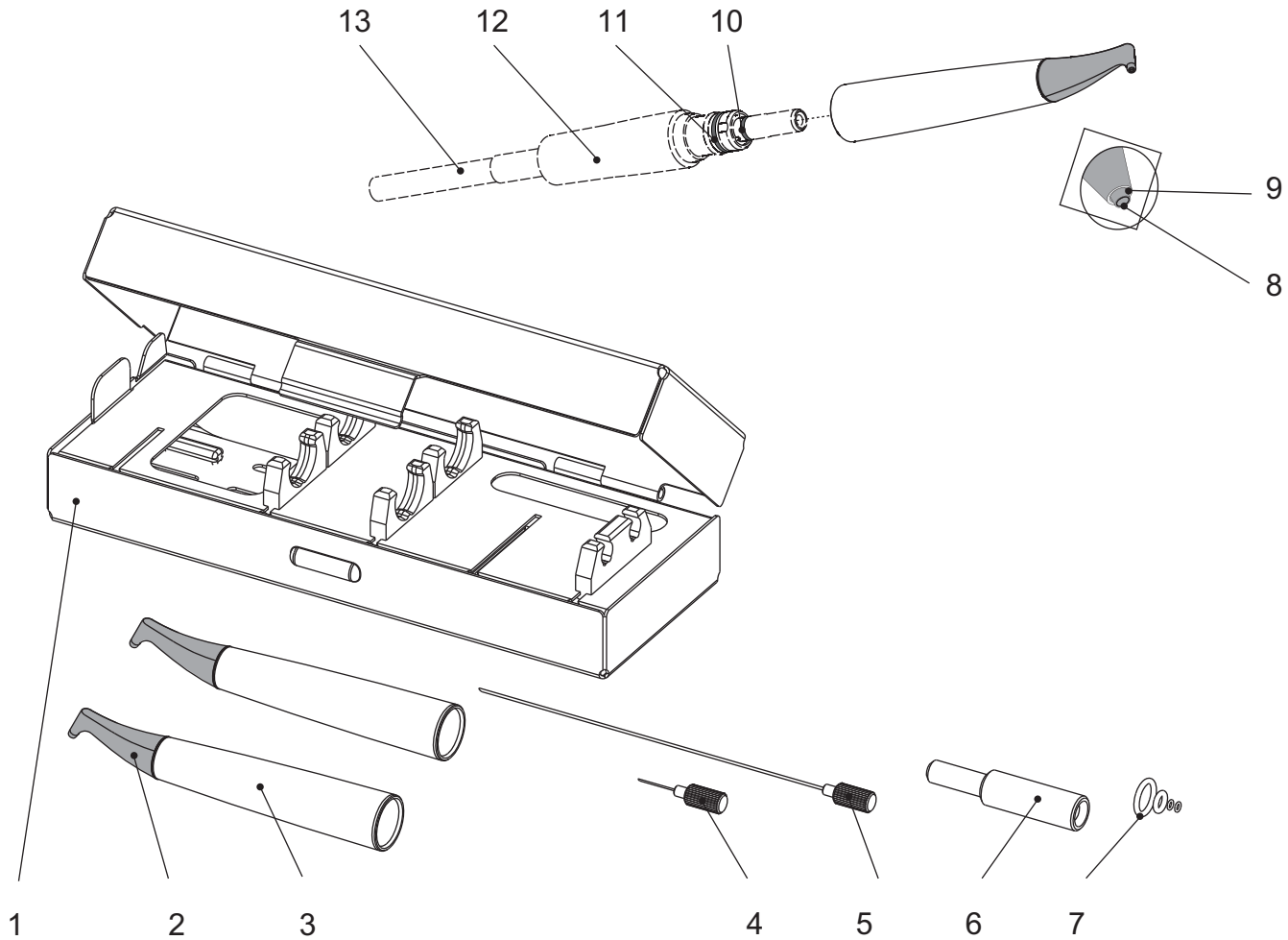EMS 
ELECTRO MEDICAL SYSTEMS

COMPONENTS

1. SystemBox AIR-FLOW®
2. Nozzle
3. Handpiece
4. Short cleaning needle
5. Long cleaning needle
6. Needle guide
7. Maintenance set
8. Powder outlet tube
9. Water outlet port
10. Small O-ring for handpiece connection
11. Large O-ring for handpiece connection
12. Handpiece connection
13. Handpiece cord

EMS supplies this product in different configurations.

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DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

This system is for use in dentistry with EMS AIR-FLOW units.

This product functions with the powder provided by EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

Applications:

- Plaque removal for placement of sealants
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers
- Surface preparation prior to placing composite restorations
- Cleaning prior to bonding orthodontic brackets
- Effective plaque and stain removal for orthodontic patients
- Clean implant fixture prior to loading
- Stain removal for shade determination
- Plaque removal prior to fluoride treatment
- Plaque and stain removal prior to whitening procedure.

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PLEASE READ THIS BEFORE BEGINNING WORK!

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Instructions concerning the components of the system are provided separately in the relative instruction for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger
Risk of injury

Caution
Risk of property damage or environmental harm

Please note
Useful additional information and hints

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Yetki verme

About this manual

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly in your approved EMS repair center or with your dealer.

We wish you lots of success!

EMS

TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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CONTENTS

Preparing for treatment.....	2
Connecting the handpiece.....	2
General hints for treatment.....	4
Contraindication.....	4
Information about powders.....	4
Wear a mask and eye protection.....	6
Treatment.....	10
Handling and setting water/air.....	12
Working technique.....	14
Precautions to be taken at the end of the treatment.....	18
Fluoride Application.....	20
Patient Information.....	22
Cleaning, disinfecting and sterilizing.....	24
Sterilization.....	29
Drying and connecting the sterilized handpiece.....	30
maintenance.....	32
O-ring check.....	32
Safety precautions.....	34
Storing the product when out of use for extended periods.....	36
Product Disposal.....	38
Warranty.....	38
Accessories.....	38
EMS-Service.....	40
Symbols.....	42
Technical data.....	43
Troubleshooting.....	44

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GENERAL HINTS FOR TREATMENT

Contraindication

 Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

 Patients on a low salt diet must not be treated with the powder containing sodium bicarbonate. For patients on a low salt diet use the powder without sodium bicarbonate provided by EMS.

Information about powders

 The aromatic powder provided by EMS can lead, in specific cases, to allergic reactions to the flavour. For patients known to suffer such reactions, use the powder without flavour provided by EMS.

 Do not direct the jet of powder toward fillings, or crown and bridgework as this could damage the restorations.

 The powder containing sodium bicarbonate provided by EMS can be used only for supra-gingival application. For sub-gingival application, please use the specific powder provided by EMS and refer to its operation instructions.

 Never use an abrasive powder i.e. silica or alumina based. This would damage the unit.

 The powders provided by EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

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 A powder jet accidentally directed into the eye can cause severe ocular injury. During treatment, we strongly advise to all persons concerned dentist, hygienist, patient, to wear eye protection.

 To limit the risk of bacterial or viral infections and the powder inhalation, we recommend to the dentist and the hygienist to wear a safety mask of protection.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

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☞ Patient mouthwash during at least 30 seconds with the BacterX® pro* solution will greatly prevent bacteria development during the treatment.

* BacterX® pro is not available in the USA.

☞ We strongly recommend applying Vaseline® on the patient's lips in order to avoid their getting dried and chapped.

☞ The cotton rollers stop the flow of saliva, part the lips and protect the gums.

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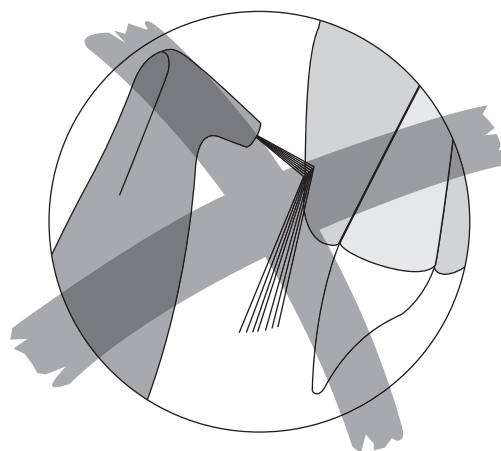
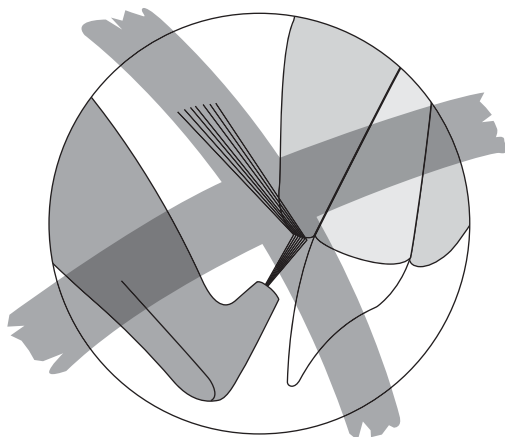
 The air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

 Proceed with small circular motions for the treatment. Polish all dental surfaces after the treatment by setting the water flow rate to maximum

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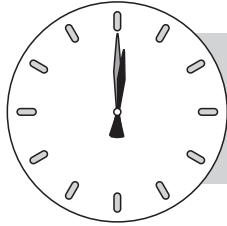
 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any more natural protection with respect to coloring.

 Tell the patient that during 2 to 3 hours following treatment, he should neither smoke, nor consume food or drinks which could strongly color the teeth (tea, coffee...).

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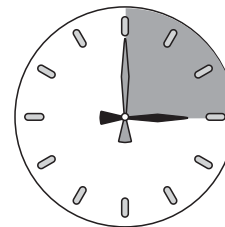
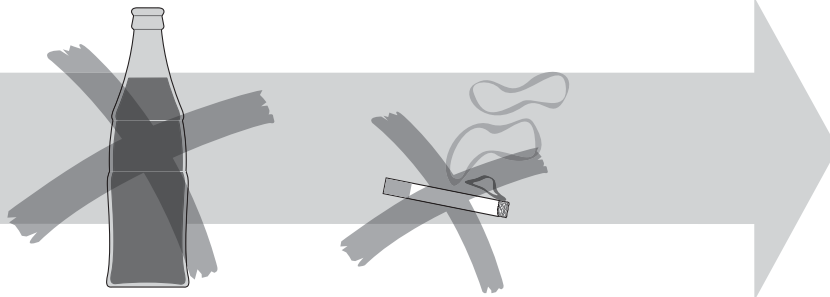
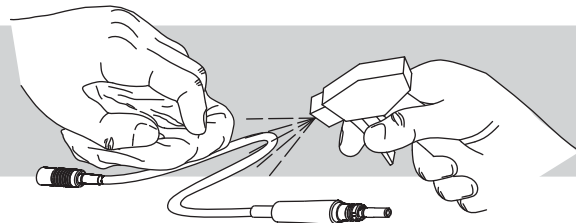
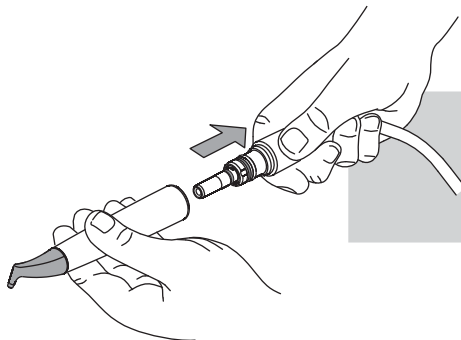
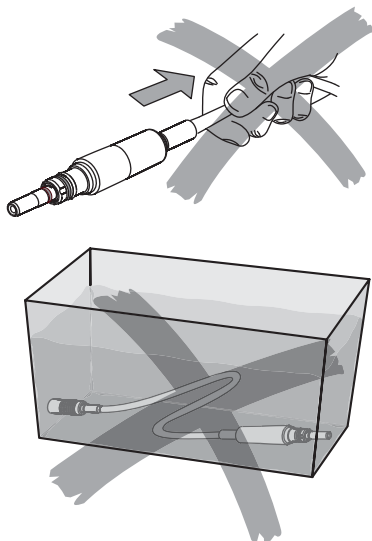
CLEANING, DISINFECTING AND STERILIZING

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⚠ Clean the handpiece and the cord only with an alcohol-based, commercially available (ethanol, isopropanol), colourless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

Remove the residual powder present in the tubes by using the cleaning needles. Proceed with precaution and do not use force because the needles can easily break. Use only the tools which are provided to you.

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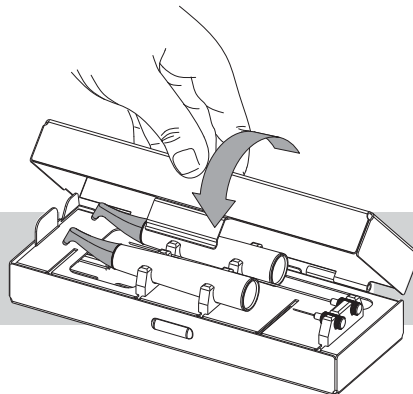
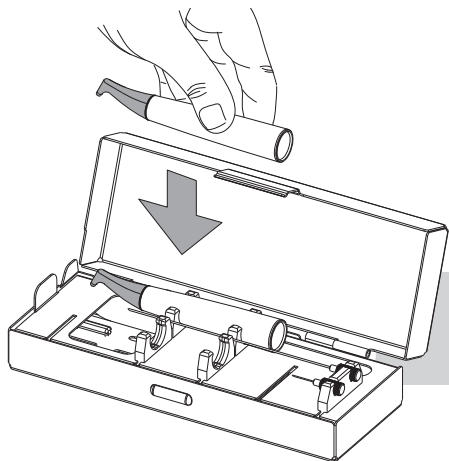
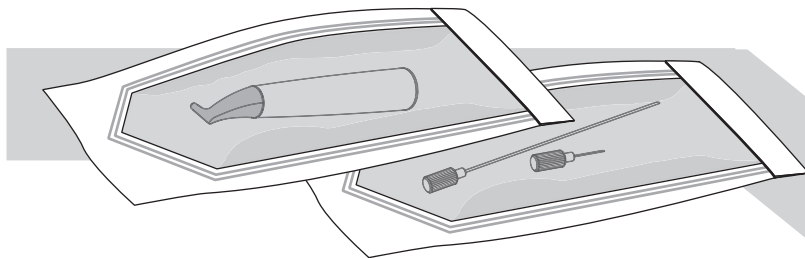
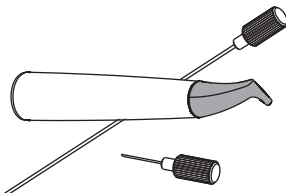
135°C
SSS

Sterilization

 After sterilizing without a sterilization bag, stains may appear on the SystemBox.

 After use, always sterilize the handpiece only in the autoclave at 134°C (135°C maximum) for at least 3 minutes.

 For sterilizing, please refer to the present-day rules in your country.

Put the accessories in sterilization bags or use the EMS SystemBox. The SystemBox allows the conditioning of the accessories for sterilizing. Different configurations are available, please ask your dental dealer. To maintain the sterility when stocking the equipment, you can put the SystemBox in a sterilization bag.

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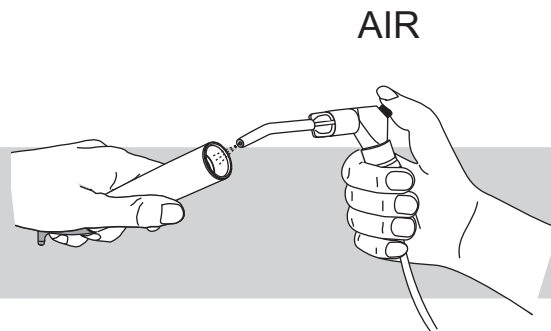
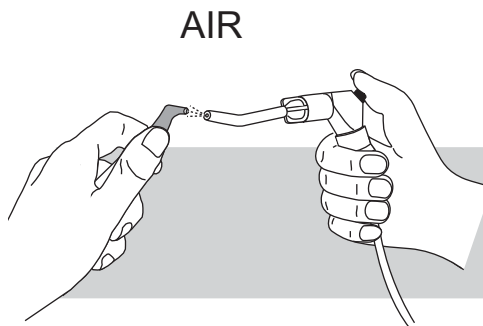
Drying and connecting the sterilized handpiece

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
⚠ After sterilization, moisture may be present in the handpiece. Blow the interior of the handpiece with dry compressed air. Make sure the handpiece connection of the cord is dry.

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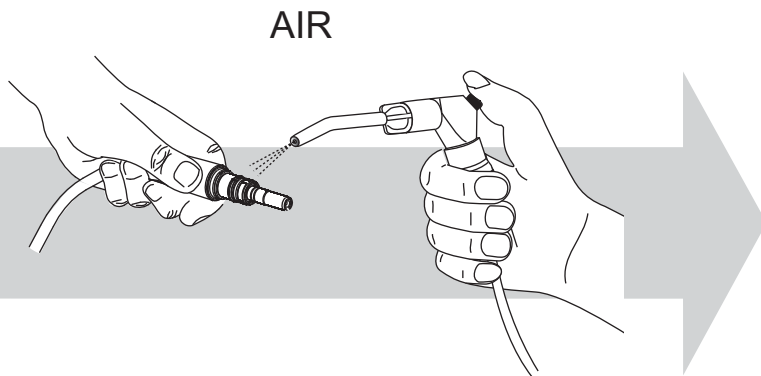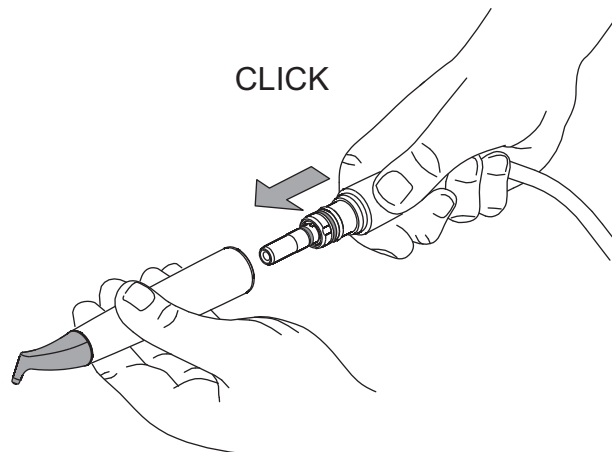
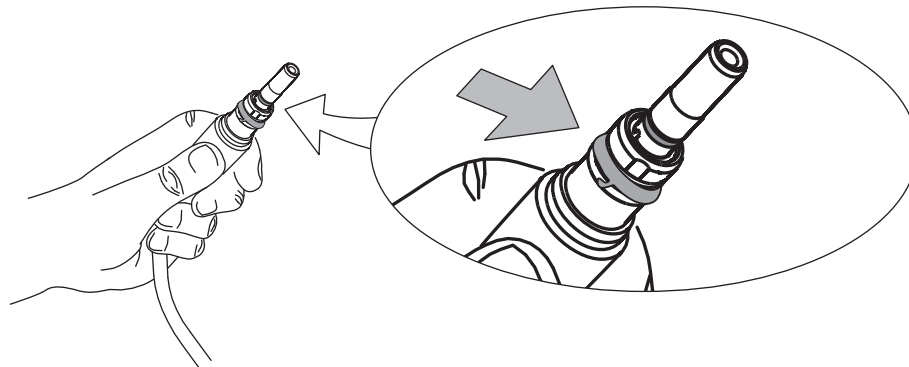
 If an O-ring is damaged or worn out, it must be replaced immediately.

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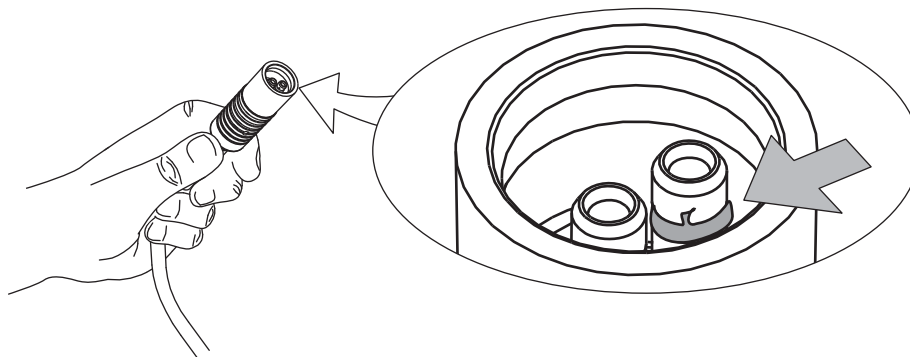
SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Before each use, disinfect, clean and sterilize the different pieces and accessories of this product. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Use for the intended purpose only: Before using the product, make sure that you have studied and understood the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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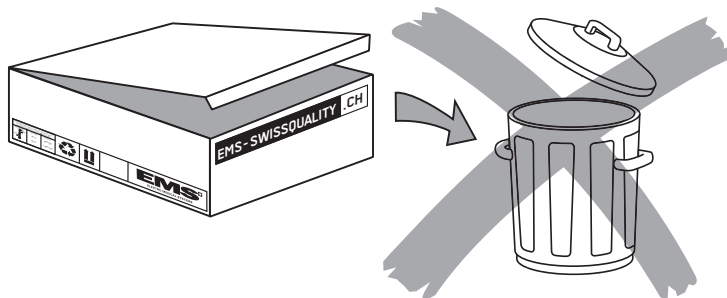
STORING THE PRODUCT WHEN OUT OF USE FOR EXTENDED PERIODS

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Disinfecting, cleaning and sterilizing"
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

 The powder should not be stored near acids or heat sources

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PRODUCT DISPOSAL

The product, its accessories and its packaging do not contain any substances that are dangerous for the environment.

Should you wish to definitively dispose the product, please comply with the regulations which apply in your country.

WARRANTY

The warranty is valid from the date of purchase of your product for the period indicated in the Registration Card.

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

DESCRIPTION

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW® Prophylaxis System
Classification 93/42 EC	Class IIa
Operating conditions	+10°C to +40°C 30% to 75% relative humidity
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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TROUBLESHOOTING

Symptom	Solutions
No water jet coming from the handpiece when the footpedal is activated	<ul style="list-style-type: none">• Check the water adjustment of your unit• Check the connections between the unit, the cord and the handpiece• Check the water supply• Check for presence of water at the cord outlet port• Check for presence of water at the system connector outlet without cord• Send the system to an approved EMS repair center
No powder/air jet coming from the handpiece when the footpedal is activated	<ul style="list-style-type: none">• Check the compressed air supply• Check for presence of powder/air at cord outlet port without handpiece• Check for presence of powder/air at the system connector outlet without cord• Eliminate residual powder present in the channels using cleaning needles• Check the quantity of powder in your unit• Send the system to an approved EMS repair center
Water leakage between handpiece cord and handpiece	<ul style="list-style-type: none">• Check the condition of the handpiece connecting O-rings• Send the system and the handpiece cord to an approved EMS repair center
The handpiece efficiency diminishes	<ul style="list-style-type: none">• Check the quantity of powder in your unit• Eliminate residual powder present in the channels using cleaning needles• Send the system and the handpiece cord to an approved EMS repair center

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e-mail: info@ems-dent.de

SPAIN

EMS Electro Medical Systems España SL

Bernardino Obregón 14 bis

E-28012 Madrid

Tlf. +34 91 528 99 89

Fax +34 91 539 34 89

e-mail: ems-es@arrakis.es

FRANCE

EMS France Sarl

23, Av. Louis Bréguet

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F-78140 Vélizy Villacoublay

Tél. +33 1 34 58 03 80

Fax +33 1 34 58 03 90

e-mail: info@ems-france.fr

ITALY

EMS Italia S.p.a

Via Faravelli 5

I-20149 Milano

Tel. +39 02 3453 8111

Fax +39 02 3453 2778

e-mail: dental@ems-italia.it

USA

EMS Corporation

11886 Greenville Avenue #120

Dallas, TX 75243, USA

Tel. +1 972 690 83 82

Fax +1 972 690 89 81

e-mail: emsusa@emsdent.com

WWW.EMS-SWISSQUALITY.CH