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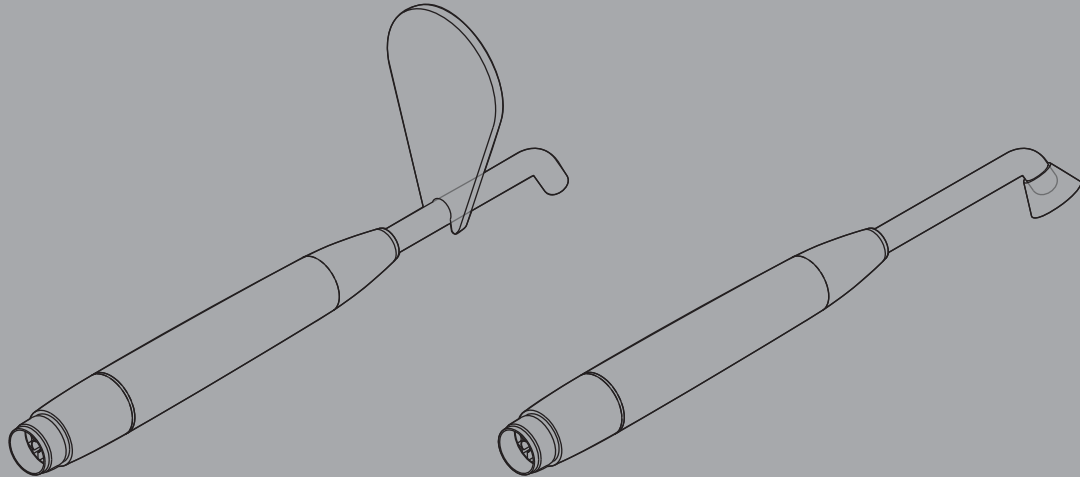
DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

This Light Curing Unit is used in dentistry for the following applications:

- Photopolymerization of photo-initiated dental materials (here after referenced as composites for easier reading of this manual).

Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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PLEASE READ THIS BEFORE BEGINNING WORK!

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger

Risk of injury

Caution

Risk of property damage or environmental harm

Please note

Useful additional information and hints

Prohibited

Authorized

About this manual

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

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We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

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EMS

CONTENTS

Installation and connection	2
Handpiece	2
Light guide	4
Settings.....	6
How to use.....	8
Switching On and Off.....	8
Treatment	10
General information	10
Cleaning, disinfecting and sterilizing	14
Handpiece and eye protection screen	14
Light guide and eye protection sleeve	16
Periodic maintenance and controls	18
Replacing the light guide	18
Replacing the protection sleeve	18
Check the handpiece.....	18
Safety precautions.....	20
Storing the product	26
Product disposal	28
Warranty	28
Accessories	28
EMS service	30
Symbols.....	32
Electromagnetic compatibility	36
Technical data.....	38
Troubleshooting.....	42

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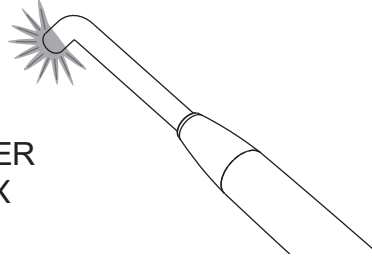
Ustawienia

POWER
MAX

Only for light curing units compatible with an EMS Piezon Scaler connection:

 To ensure a correct function of the Light Curing Unit, apply the following:

- Select "Dry Work" or set the water flow of your Piezon Scaler to zero.
- Do not select the "ENDO" mode.

 Use the ultrasonic power setting of your Piezon Scaler to adjust the intensity of the Light Curing Unit to maximum.

 For further information regarding adjustment procedures, please refer to the operating instructions of the device or dental unit.

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TREATMENT

General information

Light Curing Units transfer energy in form of light to the tooth and to the composite. Therefore, heat is always applied to the tooth and composite during the curing process. The amount of heat transferred to the tooth is proportional to the curing process duration and curing light intensity.

 Do not hold the Light Curing Unit against one given tooth for more than 40 uninterrupted seconds.

 The purpose of the Light Curing Unit is to cure layers thinner than 2 mm. Please refer to recommendations from the composite manufacturer.

 The light intensity decreases as the distance between the tip of the Light Curing Unit and the composite increases. Position the tip of the light guide as close as possible to the composite without direct contact.

 For children or patients having heat sensitivity, reduce the exposition time. Ask your patient to express any heat sensitivity during the treatment.

 Avoid directing the light towards the eyes, soft tissues or unprotected skin.

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 Use a rubber or liquid dam to protect the gingiva during treatment.

 Because of the high light intensity of this unit, it is definitely required that the dentist and the assistant wear orange protective glasses, and/or use the eye protection screen and eye protection sleeve during operation.

 Use only light curing materials that can be photo-initiated in the specified wavelength range of the Light Curing Unit. For more information see the "Technical data" chapter of this product and the specifications of the composite manufacturer.

 To maintain the performance of the light guide, avoid as much as possible any contact with the composite.

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
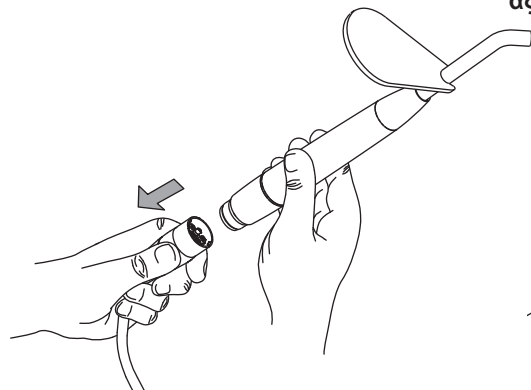
CLEANING, DISINFECTING AND STERILIZING

Handpiece and eye protection screen

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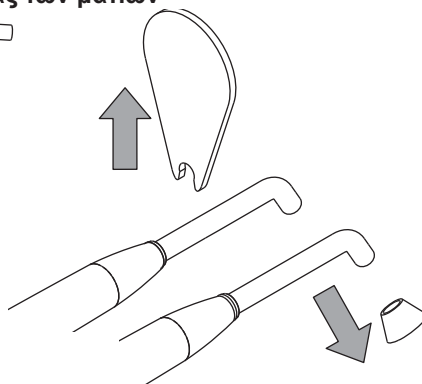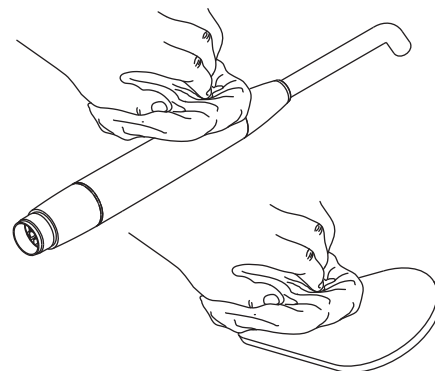
⚠ Clean the handpiece, the handpiece cable, and the eye protection shield with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage the surface.

⚠ Do not immerse the Light Curing Unit into disinfectant, as this would damage it.

⚠ The Light Curing Unit and the eye protection screen cannot be sterilized.

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
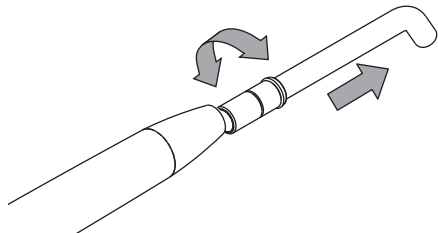
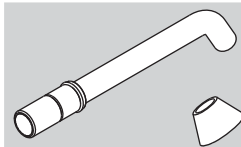
Light guide and eye protection sleeve

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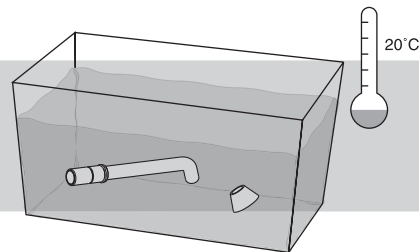 Disinfect with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant.

 Rinse the disinfected glass light guide and the protection sleeve under running water before sterilization.

 After each use, sterilize the glass light guide and the protection sleeve in an autoclave at 134°C (135°C maximum) for at least 3 minutes.

 For sterilizing, please refer to the current rules in your country.

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PERIODIC MAINTENANCE AND CONTROLS

Replacing the light guide

Regularly check the condition of the light guide. Replace if it is scratched, cracked or chipped.

 To maintain the performance of the light guide, avoid as much as possible, any contact with the composite.

 Repeated sterilization can reduce the light guide performance. Check the light intensity regularly with a radiometer.

 You can obtain a replacement light guide from your dental equipment dealer.

Replacing the protection sleeve

 Replace the eye protection sleeve after 50 sterilizations cycles.

Check the handpiece

 Check the light intensity of the handpiece regularly with a radiometer.

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SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 **Contra-indication (only for light curing units compatible with a Piezon Scaler connection):** High frequencies may prevent cardiac pacemakers proper function. Therefore, we recommend that patients with a cardiac pacemaker should not be treated with this product.

 **Use for the intended purpose only:** These operating instructions must be read and understood before using the product. This also applies to any device used in combination with this unit. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 **Risk of explosion:** Do not use this product in the presence of inflammable anaesthetics or gases.

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 Use only EMS original parts: Using parts or accessories other than those specified or sold by EMS as replacement parts may result in decreased efficiency of this product.

 This product must be used only by trained and qualified personnel.

 The light curing units compatible with the Piezon Scaler connection can be only used with EMS ultrasound devices or dental units equipped with a Piezon kit "EMS".

 Before each use, clean, disinfect and sterilize the different pieces and accessories of this product. Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 The patient or the user may get hurt or infected when using a chipped or scratched light guide.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

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 Have this product exclusively repaired by an approved EMS repair center.

 This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated physiological effects.

 Inadequate light emission can cause incomplete polymerization of the composite.

 When the curing time required for a particular restoration is not known, a test sample should be created using an extracted tooth or commercially available mould.

 People with a medical history indicating photo biological reactions (including people suffering from urticaria or erythropoietic photoporphyria) or using photosensitization drugs should not be exposed with the Light Curing Unit.

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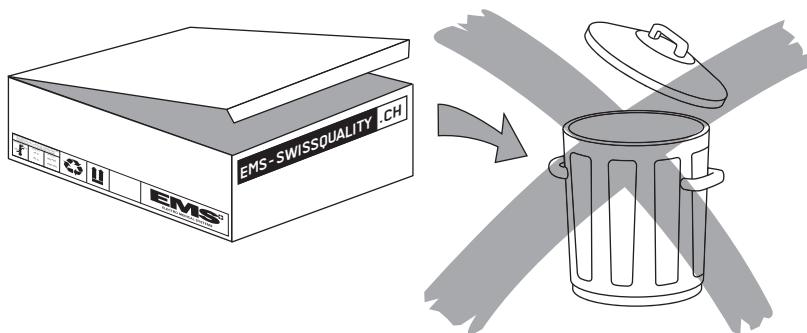
 Keep the original packaging until the product is to be disposed of permanently. You may use it for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing",
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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PRODUCT DISPOSAL

The product, its accessories and its packaging do not contain any substances that is dangerous for the environment.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

WARRANTY

The warranty is valid from the date of purchase of your product for the period indicated in the Registration Card.

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damage due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before shipping your product with all its accessories, please clean, disinfect and sterilize as described in the operating instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This will make processing easier.

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ELECTROMAGNETIC COMPATIBILITY

 This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the "Electromagnetic compatibility" notice.

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" must be followed.

 This product should not be used adjacent to or stacked with an other unit. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

 The use of accessories and cables other than those specified or sold by EMS as replacement parts, may result in increased emissions or decreased immunity of this product.

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TECHNICAL DATA

DESCRIPTION

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland	
Model	DEEP BLUE® Piezon compatible system	DEEP BLUE® Dental unit kit
Classification EN 60601-1 <i>*with inserted light guide</i>	- Applied part, Type BF - IP 55*	- Applied part, Type BF - IP 55*
Classification 93/42/EEC	Class I	Class I
Mode	Intermittent operation: 10x (20 sec. ON + 10 sec. OFF) + 15 min. OFF	
Supply voltage	max. 250 VAC	10 - 24 VAC ±10% 13.5 - 32 VDC ±10%
Supply frequency	24 - 32 kHz	For AC: 60/50 Hz
Power consumption	max. 7 VA	max. 6 VA
Fuse		1AT, 250 V (IEC 60127)
Light radiation wave length interval at maximum power	Between 430 and 480 nm	
Weight (without light guide)	~ 0.065 kg	
Size (without light guide)	Ø 20 x 220 mm	Ø 20 x 205 mm
Light density	~1000 mW/cm²	
Operating conditions	+10°C to +30°C 30% to 75% relative humidity	
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa atm. pressure	

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TROUBLESHOOTING

Type of problem	Solutions
The Light Curing Unit does not light up	<ul style="list-style-type: none">• Verify Light Curing Unit connections• Verify foot pedal connection & function• Verify that the power setting is set at MAX power• Contact your EMS Repair Center
Many continuous beeps, the Light Curing Unit does not light up	<ul style="list-style-type: none">• Verify Light Curing Unit connections• Temperature of the Light Curing Unit is too high, let it cool down for some time• Foot pedal pressed, release foot pedal contact• Power/intensity too low, increase the power to maximum• Defective Light Curing Unit, return it to your EMS Repair Center
Polymerization too long	<ul style="list-style-type: none">• Set/Increase the power to maximum• Replace the light guide• Contact your EMS Repair Center
Low light intensity	<ul style="list-style-type: none">• If device is so equipped, verify that the "ENDO" program is NOT set• Set/Increase the power to maximum• Replace the light guide• Defective Light Curing Unit, return it to your EMS Repair Center
Motor/pump runs with the Light Curing Unit	<ul style="list-style-type: none">• "DRY WORK" function is NOT set• Verify the program selected with the Light Curing Unit• For PM400, select the switch to ENDO and press till the second step of the foot pedal
The Light Curing Unit connection to the cable is difficult	<ul style="list-style-type: none">• Verify the O-ring on the connector• Replace the O-ring using a plastic tweezer
Water spray at disconnection of the Light Curing Unit	<ul style="list-style-type: none">• The water regulation is NOT set at minimum• The Light Curing Unit is NOT used with the "DRY WORK" function

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e-mail: info@ems-dent.de

SPAIN

EMS Electro Medical Systems España SL

Bernardino Obregón 14 bis

E-28012 Madrid

Tlf. +34 91 528 99 89

Fax +34 91 539 34 89

e-mail: ems-es@arrakis.es

FRANCE

EMS France Sarl

23, Av. Louis Bréguet

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F-78140 Vélizy Villacoublay

Tél. +33 1 34 58 03 80

Fax +33 1 34 58 03 90

e-mail: info@ems-france.fr

ITALY

EMS Italia S.p.a

Via Faravelli 5

I-20149 Milano

Tel. +39 02 3453 8111

Fax +39 02 3453 2778

e-mail: dental@ems-italia.it

USA

EMS Corporation

11886 Greenville Avenue #120

Dallas, TX 75243, USA

Tel. +1 972 690 83 82

Fax +1 972 690 89 81

e-mail: emsusa@emsdent.com

WWW.EMS-SWISSQUALITY.CH