
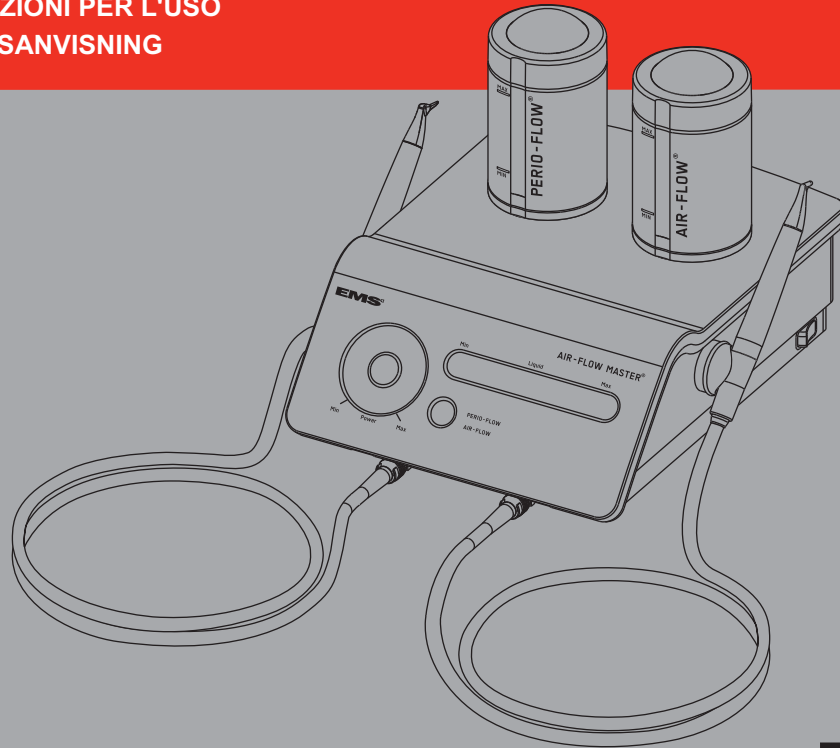
OPERATION INSTRUCTIONS
 MODE D'EMPLOI
 BEDIENUNGSANLEITUNG
 INSTRUCCIONES DE EMPLEO
 ISTRUZIONI PER L'USO
 BRUKSANVISNING

AIR-FLOW MASTER®

EMS 
 ELECTRO MEDICAL SYSTEMS

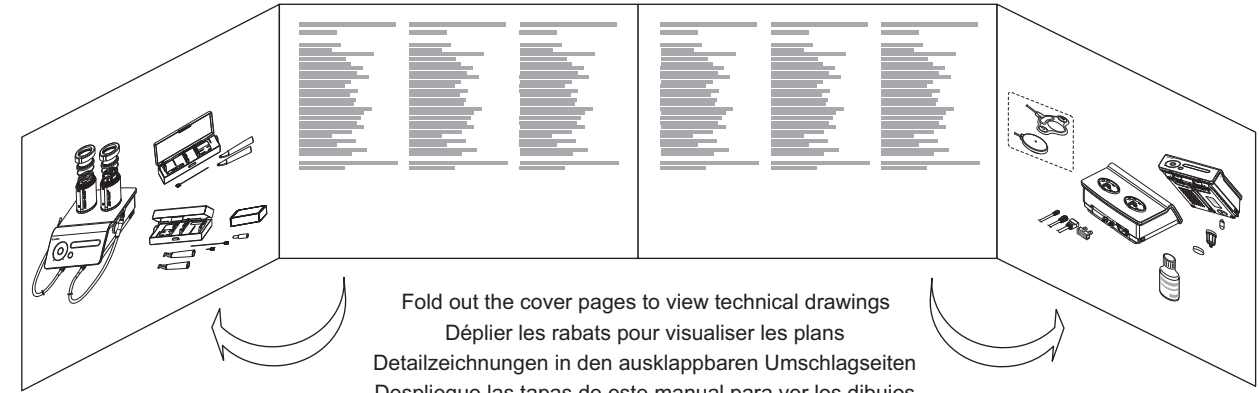Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

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TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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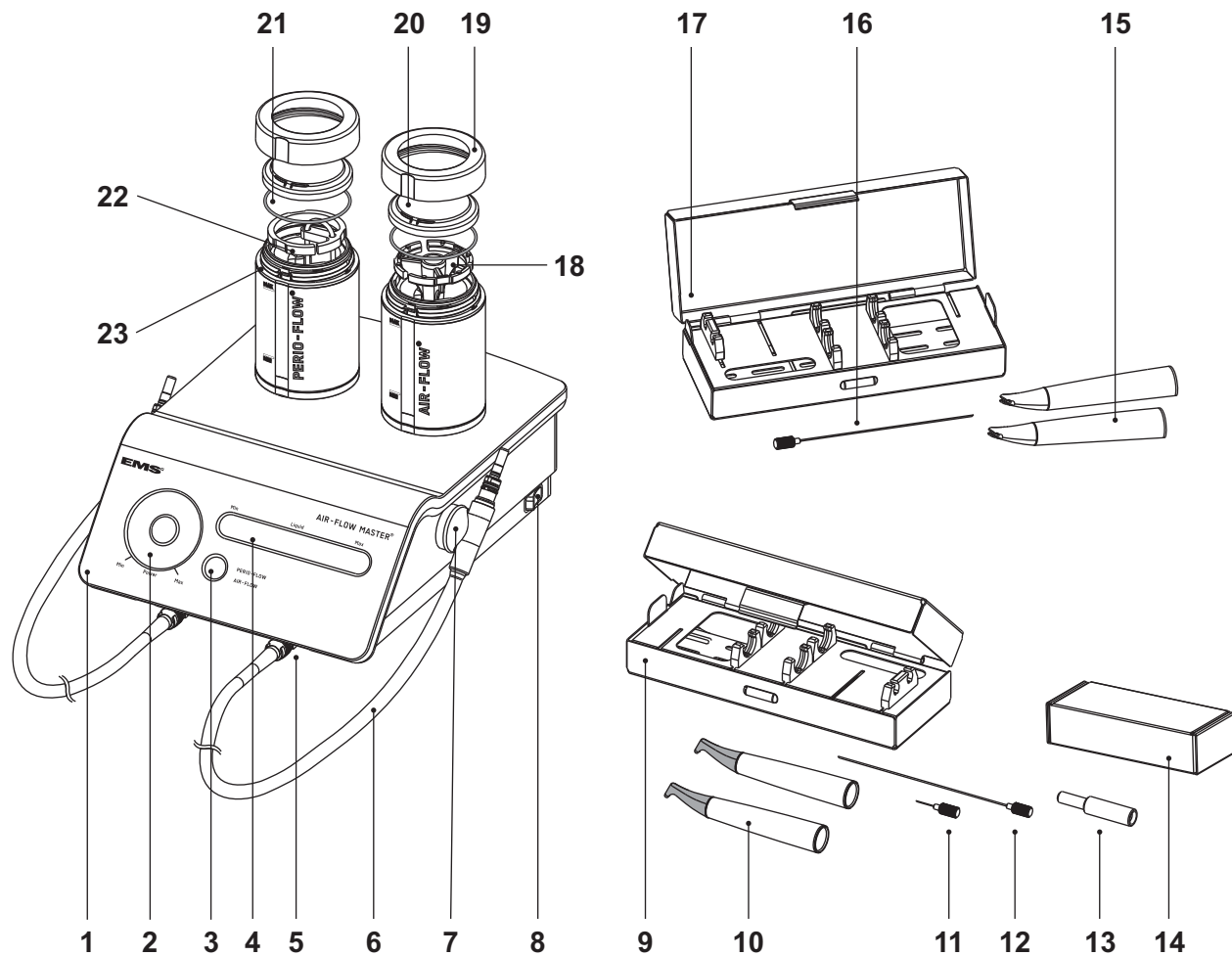
COMPONENTS

- 1 Control panel
- 2 Power setting (air/powder)
- 3 Working mode selection
- 4 Water flow rate adjustment
- 5 Connector for the handpiece cord
- 6 Handpiece cord
- 7 Magnetic handpiece support
- 8 Purge switch
- 9 AIR-FLOW® SystemBox
- 10 AIR-FLOW® handpiece
- 11 Short cleaning needle
- 12 Long cleaning needle
- 13 Needle guide
- 14 Maintenance Set
- 15 PERIO-FLOW® handpiece
- 16 Cleaning needle
- 17 PERIO-FLOW® SystemBox
- 18 AIR-FLOW® Powder diffuser
- 19 Cap ring
- 20 Cap dome
- 21 Cap seal
- 22 PERIO-FLOW® Powder diffuser
- 23 Inter-changeable colored O-ring
- 24 Foot pedal (2 step or multifunction depending on order)
- 25 Water supply hose
- 26 Compressed air supply hose
- 27 Power cable
- 28 Fuses
- 29 Water hose connector
- 30 Air hose connector
- 31 Power socket
- 32 Power switch
- 33 AIR-FLOW® CLASSIC powder bottle
- 34 Cap
- 35 Air filter housing
- 36 Filter cartridge

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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EMS Corporation