OPERATION INSTRUCTIONS

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
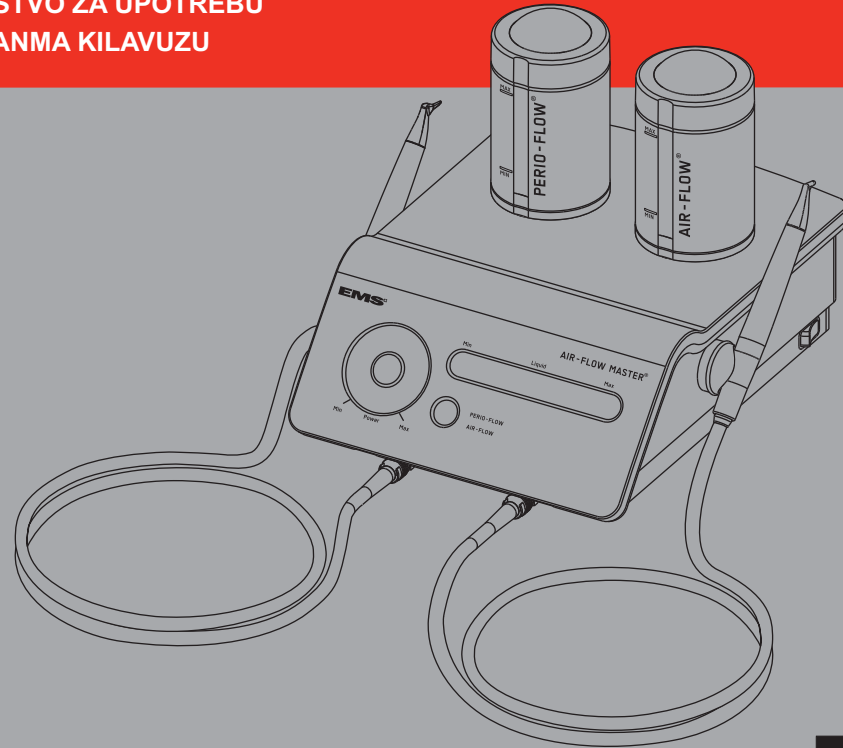
KULLANMA KILAVUZU

AIR-FLOW MASTER®

Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

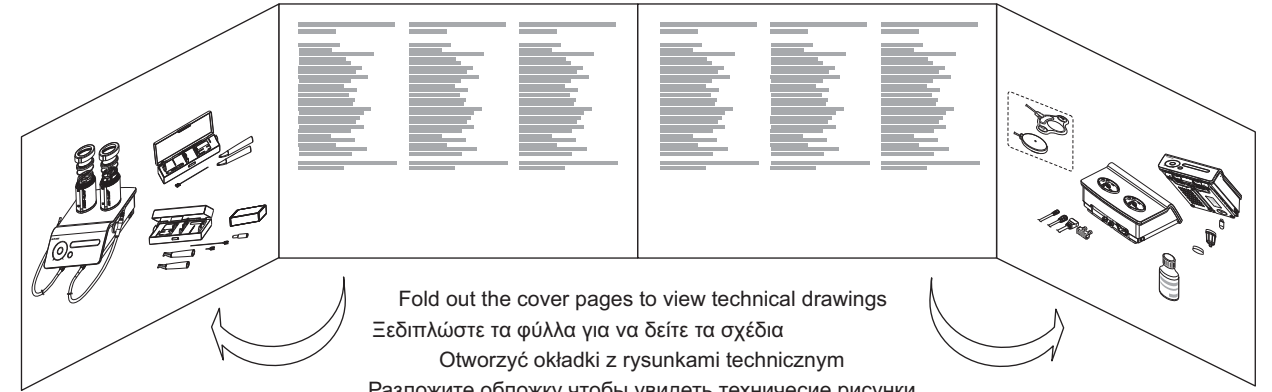EMS 
ELECTRO MEDICAL SYSTEMS

Fold out the cover pages to view technical drawings
Ξεδιπλώστε τα φύλλα για να δείτε τα σχέδια
Otworzyć okładki z rysunkami technicznym
Разложите обложку чтобы увидеть технические рисунки
Korice uputstva previti prema spolja da bi se videli tehnički crtežii
Teknik detayları görmek için kapak sayfalarını çeviriniz

TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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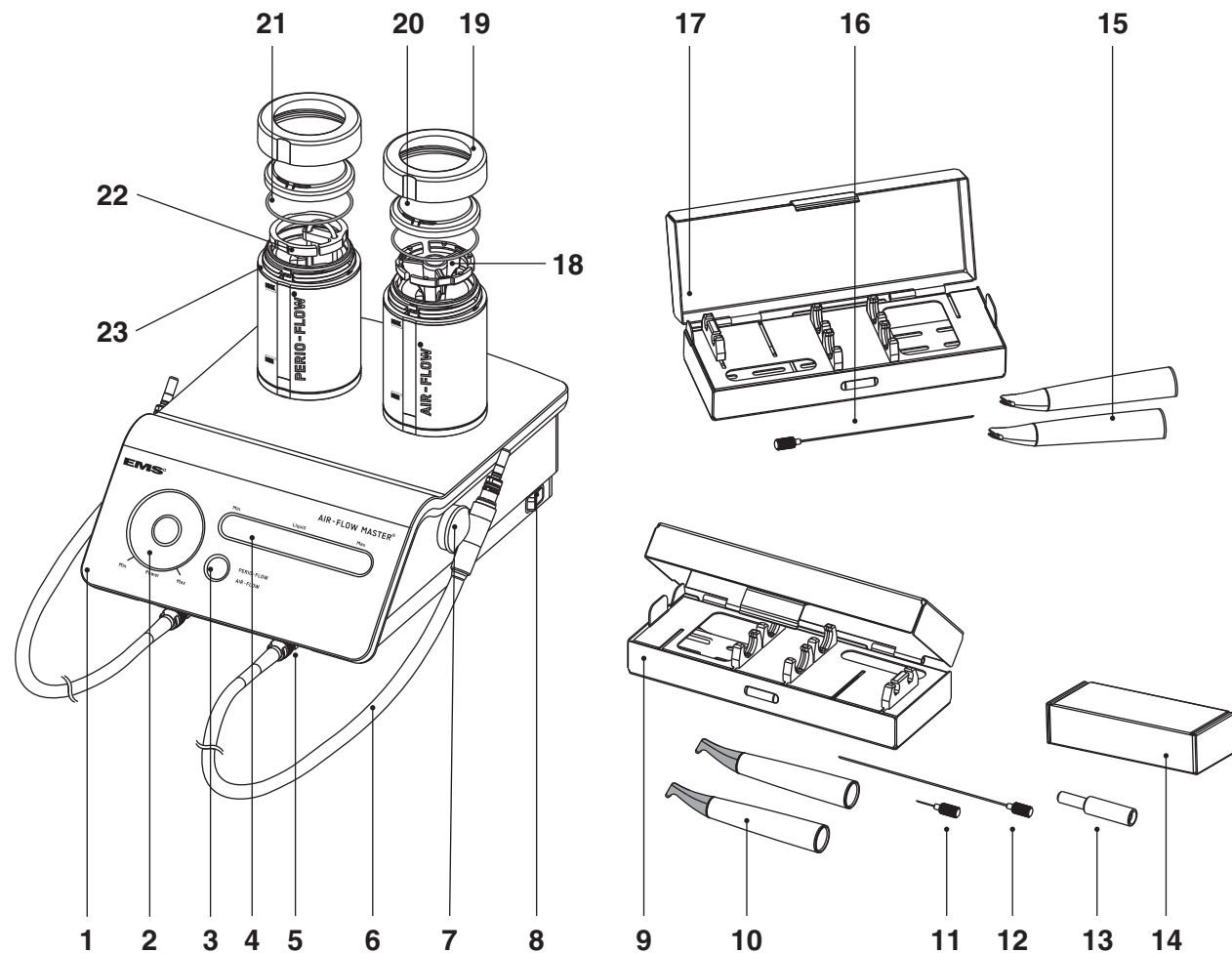
COMPONENTS

- 1 Control panel
- 2 Power setting (air/powder)
- 3 Working mode selection
- 4 Water flow rate adjustment
- 5 Connector for the handpiece cord
- 6 Handpiece cord
- 7 Magnetic handpiece support
- 8 Purge switch
- 9 AIR-FLOW® SystemBox
- 10 AIR-FLOW® handpiece
- 11 Short cleaning needle
- 12 Long cleaning needle
- 13 Needle guide
- 14 Maintenance Set
- 15 PERIO-FLOW handpiece
- 16 Cleaning needle
- 17 PERIO-FLOW® SystemBox
- 18 AIR-FLOW® Powder diffuser
- 19 Cap ring
- 20 Cap dome
- 21 Cap seal
- 22 PERIO-FLOW Powder diffuser
- 23 Inter-changeable colored o-ring
- 24 Footpedal (2 step or multi-function depending on order)
- 25 Water supply hose
- 26 Compressed air supply hose
- 27 Power cable
- 28 Fuses
- 29 Water hose connector
- 30 Air hose connector
- 31 Power socket
- 32 Power switch
- 33 AIR-FLOW CLASSIC powder bottle
- 34 Cap
- 35 Air filter housing
- 36 Filter cartridge

EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes.

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EMS Corporation