Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Product Description

The AIR-FLOW® Classic (prophylaxis powder) is used solely for supragingival treatments.

This powder is for use in dentistry with the EMS airpolishing devices.

Applications

This product is recommended for:

- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Clean implant fixture prior to loading,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

Contra-indications

⚠ Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an airpolishing device. The jet of air and powder could cause respiratory difficulties.

⚠ Patients on a low salt diet must not be treated with the powder as it contains sodium bicarbonate. For these patients it is recommended to use the salt free powders also provided by EMS (see below).

⚠ The flavor contained in the powder can lead, in specific case, to allergic reactions. For patients known to suffer from such reactions, use the "Neutral" powder provided by EMS.

⚠ Do not direct the jet of powder toward fillings or crown and bridgework as this could damage the restorations.

⚠ Do not use this powder subgingivally. For this application use only the powder recommended by EMS for subgingival application.

Recommendations for the treatment

⚠ Patient and user have to wear eye protection.

⚠ User has to wear a mask.

👉 Use the high-speed evacuator of your dental unit.

👉 Apply Vaseline® on the patient's lips.

General information

- Do not exceed the powder filling limit as indicated on the container.
- This powder is sugar free.

Storage and shelf-life


- The powder should not be stored near acids or heat sources.
- Storage condition -10°C to 40°C and 10% to 95% relative humidity.
- Do not use after expiration date.

Also available:

- AIR-FLOW® Classic (available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral).
- AIR-FLOW® Soft (salt free).
- AIR-FLOW® Perio (salt free for subgingival application).

Additional safety advice for USA

Caution! Federal (USA) law restricts this unit to sale by or on the order of a dentist.

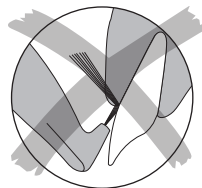👉 Note that the English version of this instruction is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

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