

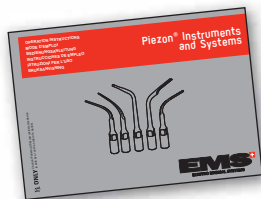
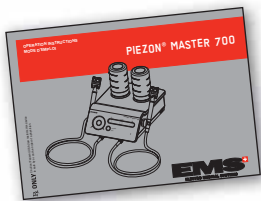
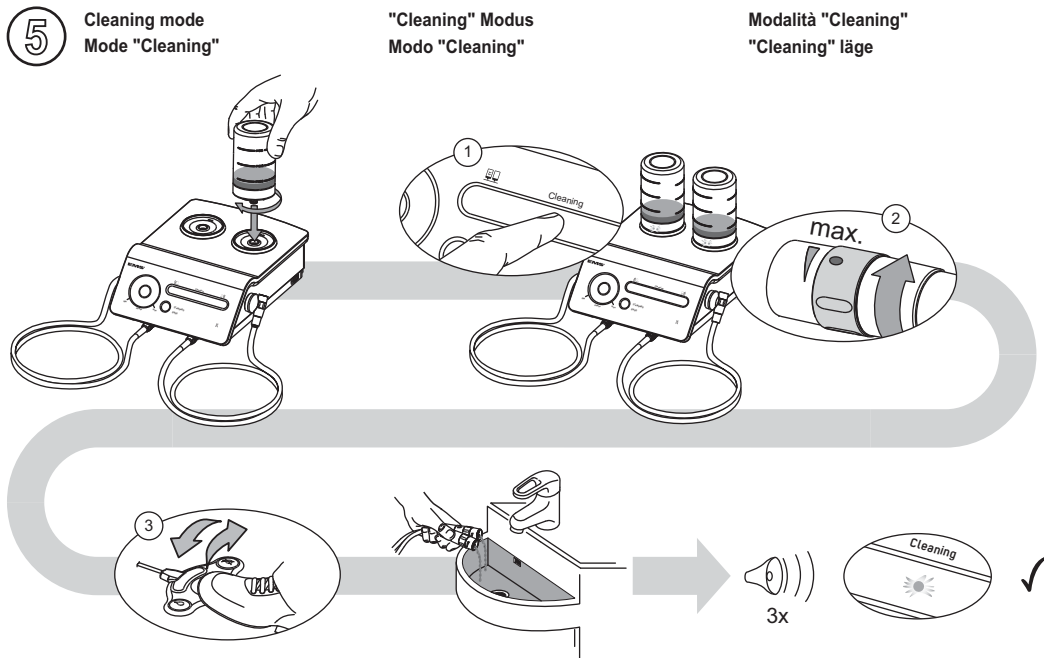


Piezon® Master 700

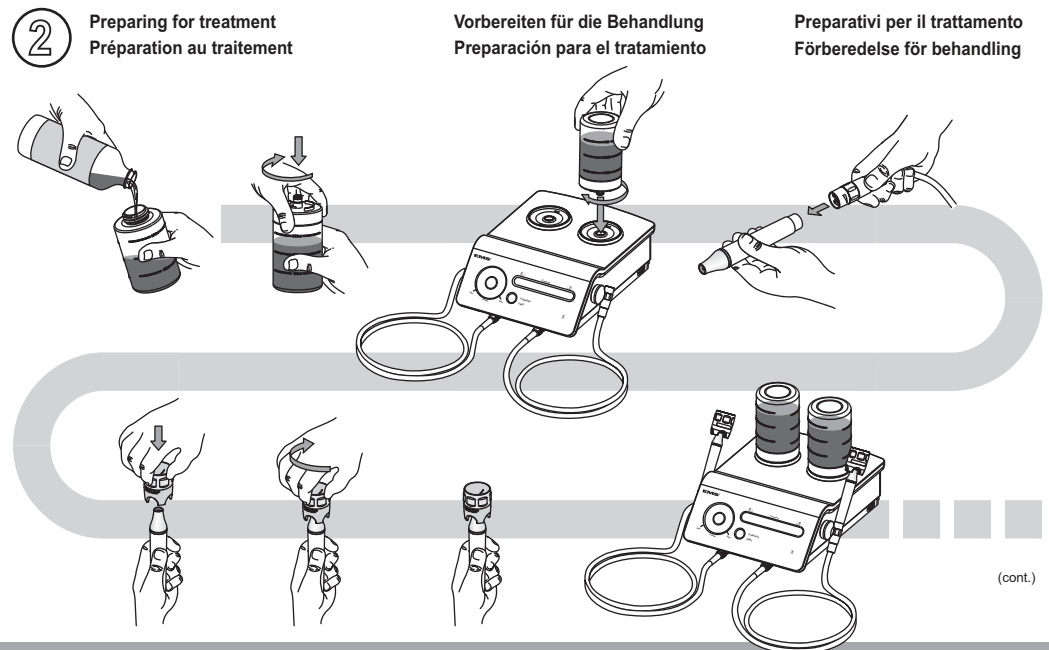

© Copyright EMS SA FB-4323 ed. 2009/01

Piezon® Master 700

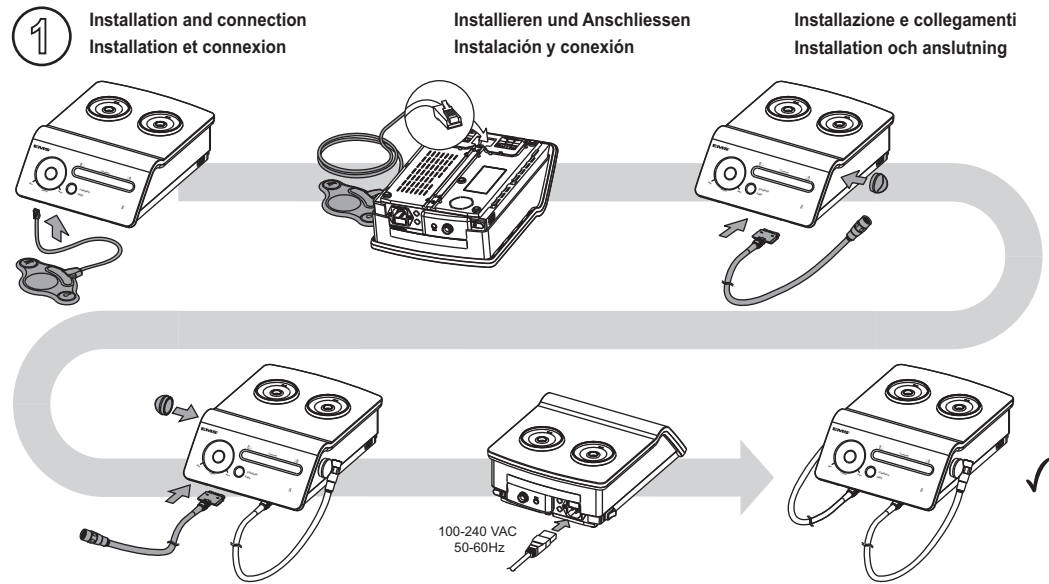


(cont.)

Read the operating instructions
Consulter le mode d'emploi
Bitte Bedienungsanleitung lesen




Ver instrucciones de empleo
Vedi istruzioni per l'uso
Läs bruksanvisningen

EMS
ELECTRO MEDICAL SYSTEMS

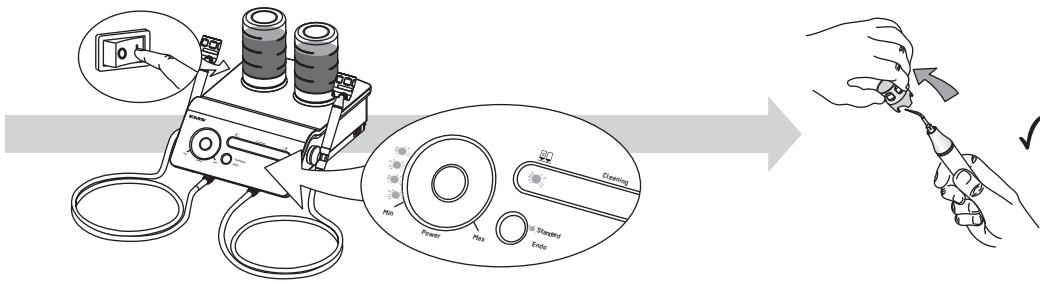


3a Working mode / Betriebsmodus / Modalità di funzionamento / Mode de travail / Modo de funcionamiento / Arbeitsmetod

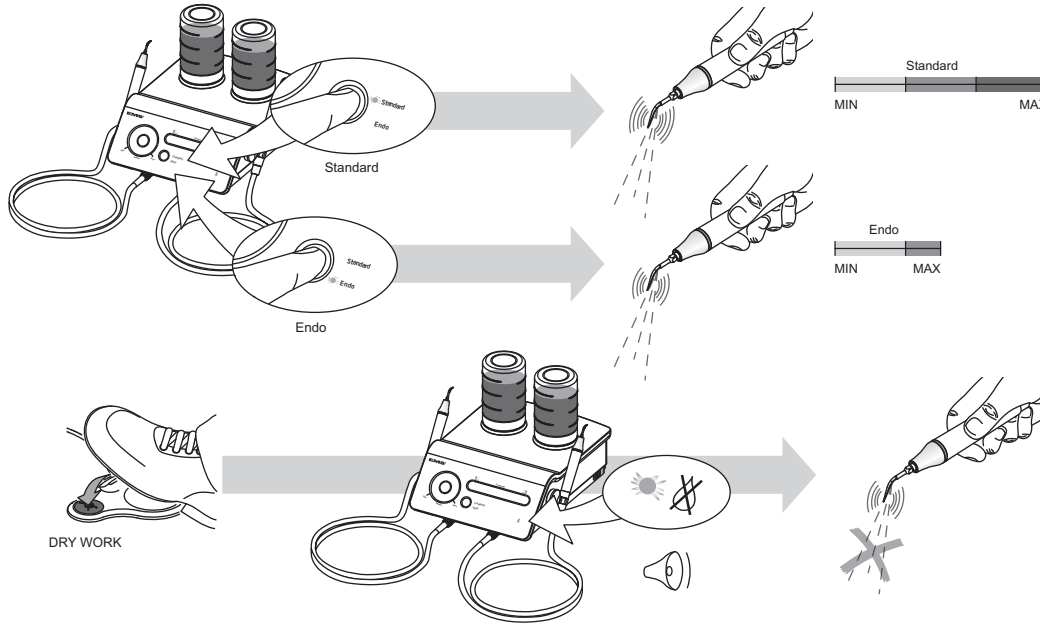

3b Multifunction foot pedal / Multifunktionsfußschalter / Multifunction pedale / Pédale multifonctions / Multifunction pedal / Multifunction pedal

	EN	FR	DE	ES	IT	SV
Standard mode						
1	Irrigation + light	Irrigation + lumière	Flüssigkeit + Licht	Irrigación + luz	Irrigazione + illuminazione	Spolning + belysning
2	Ultrasounds + irrigation + light	Ultrasons + irrigation + lumière	Ultraschall + Flüssigkeit + Licht	Ultrasonidos + irrigación + luz	Ultrasuoni + irrigazione + illuminazione	Ultraljud + spolning + belysning
3	Ultrasounds + light (DRY WORK mode)	Ultrasons + lumière (mode DRY WORK)	Ultraschall + Licht (Modus "DRY WORK")	Ultrasonidos + luz (modo "DRY WORK")	Ultrasuoni + illuminazione (Modalità "DRY WORK")	Ultraljud + belysning ("DRY WORK"-läge)
4 (+2)	Boost function + light	Fonction "Boost" + lumière	"Boost" Funktion + Licht	Función "Boost" + luz	Funzione boost + illuminazione	Effektökning + belysning
Endo mode						
1 2 3	Same as Standard mode	Identique au mode Standard	Identisch mit Standard-Modus	Igual que el modo Standard	Come in modalità Standard	Samma som Standard-läge
4	Inactive	Inactif	Inaktiv	Inactivo	Inattivo	Inaktivt

3c Bottle, power and flow rate / Flasche, Leistung und Durchflussrate / Bouteille, puissance et débit / Flasche, potencia y caudal / Flaska, effekt och flöde

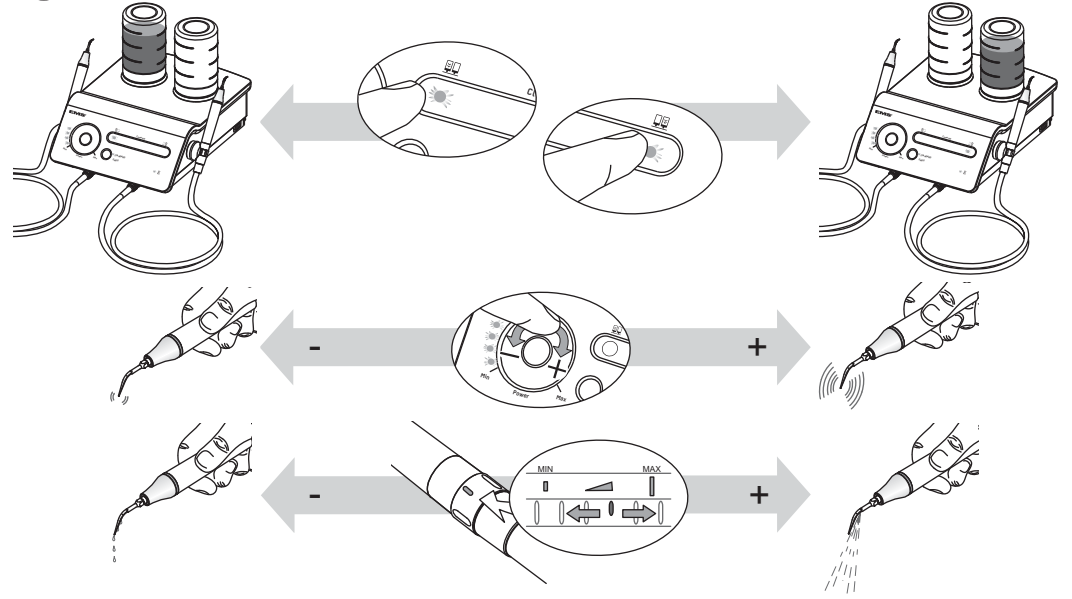

3d Light / Lumière / Licht / Luz / Illuminazione / Belysning



4 End of treatment / Fin du traitement / Beenden der Behandlung / Fin del tratamiento / Fine del trattamento / Avsluta behandlingen

